

Little Sucker[®] and Little Sucker Cover[®]

Oral and Nasal Suction Device and Protective Cover



Features and Benefits

Little Sucker

- ✓ Designed for single handed suctioning
- ✓ Thumb port for intermittent suctioning
- ✓ Soft, flexible tip similar to a bulb syringe
- ✓ Not made with rubber latex or plasticizer DEHP

Little Sucker Cover

- ✓ Designed to protect the Little Sucker from damage, debris and external contaminants
- ✓ Bright blue color for better visibility
- ✓ Noise reducing port plug
- ✓ Not made with rubber latex or plasticizer DEHP

Item #	Item	Qty/Unit
602-N204	Preemie	50/box
602-N204C	Preemie w/ Cover	50/box
602-N204TP	Preemie 2-piece*	50/box
602-N205	Standard	50/box
602-N205C	Standard w/ Cover	50/box
602-N205TP	Standard 2-piece*	50/box
602-N215	Cover	50/box

**2-piece Little Suckers are not compatible with the Cover*



Contact your local Tri-anim Account Manager
for more information or visit: tri-anim.com

Little Sucker® and Little Sucker Cover®

Oral and Nasal Suction Device and Protective Cover

Indications for Use

The Little Sucker is indicated for use as a tip for suction tubing to aspirate fluids from the mouth and nares. The Little Sucker Cover is indicated for use to protect the Little Sucker.

Directions for Use

Step 1 (if applicable)

Unplug thumb port and slide Little Sucker out of Little Sucker Cover.

Step 2

Connect barbed end of Little Sucker to suction line.

Step 3

Set suction to appropriate pressure per hospital protocol or physician recommendation.

Step 4

Block thumb port with thumb to begin suctioning.

Step 5

Suction mouth or nares with intermittent suctioning.

Note: For intermittent suctioning, unblock thumb port to stop suctioning. Block thumb port again to continue suctioning.

Step 6

Suction sterile water through Little Sucker after each use.

Step 7 (if applicable)

Reinsert Little Sucker into Little Sucker Cover.

Step 8 (if applicable)

Insert port plug into Little Sucker thumb port.

Step 9

Replace Little Sucker and/or Little Sucker Cover every 24 hours or per hospital protocol, whichever is sooner.

Product Materials

- Little Sucker: TPE (thermoplastic elastomer)
- Cover: PVC
- Made with natural rubber latex: No
- Made with plasticizer DEHP: No

Size

602-N204 Preemie

Length: 106 mm

Tip opening diameter: 2 mm

Thumb port opening diameter: 5 mm

602-N205 Standard

Length: 120 mm

Tip opening diameter: 3 mm

Thumb port opening diameter: 5 mm

602-N204TP Preemie

Length: 116 mm

Tip opening diameter: 2 mm

Thumb port opening diameter: 6 mm

602-N205TP Standard

Length: 130 mm

Tip opening diameter: 3 mm

Thumb port opening diameter: 6 mm

Cover

Length: 90 mm

Packaging

- Little Sucker and Little Sucker Cover packaged together with the cover on.
- Package type: Polyethylene bag
- Dispenser box: Yes

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

Cautions & Warnings

- Federal law restricts this device to sale by or on the order of a physician.
- For oral and nasal use only.
- Inserting the Little Sucker too far into the mouth or nares can cause tissue trauma.
- The Little Sucker and Little Sucker Cover should remain free of debris.
- Single patient use only.



Contact your local Tri-anim Account Manager
for more information or visit: tri-anim.com