



Rx Only   **REF 301-1979**

–Non-sterile
–Not made with
natural rubber latex

Instructions for Use

Intended Use: Flusso™ TFI allows for temporary interruption of gas flow to facilitate circuit disconnection.

Indications for Use: Patients requiring means of artificial ventilation requiring circuit disconnection.

Directions for Use:

- Store Flusso™ TFI in original packaging until use.
- Attach Flusso™ TFI to the endotracheal tube of the patient, then attach the ventilator circuit.
- Ensure all connections are secure. 15mm ID / 15mm OD connections, device length is 5.65 cm.
- With your index finger and thumb, activate (compress) the tabs equally on the device. Compress the tabs to the point of where the tabs are flush with the sides of the device.
- Only the hand / fingers of a trained clinician should be used to activate Flusso™ TFI.
- Inspect the device on a routine basis to ensure proper use and function. Should the device become soiled or damaged, do not use. Discard and replace with a new one.
- Discard the adapter when replacing the circuit according to facility guidelines, with a maximum usage of 7 days.

Caution / Attention:

- Do not use any devices to try and activate, lock, or hold the compression tabs of the Flusso™ TFI.
- The activation of Flusso™ TFI should not exceed 3 seconds or as per hospital protocol.
- Use only enough force to active the compression tabs; failure to do so may allow gas flow to pass through.

Contraindications:

- Patients who cannot tolerate 1.8 cc of dead space.

Distributed by Sarnova HC, LLC's family of companies:
Bound Tree Medical, LLC, Cardio Partners, Inc.,
& Tri-anim Health Services, Inc.
5000 Bradenton Avenue
Dublin, OH 43017
855.287.2759
www.curaplex.com

Rev.3 20251202